

Multi Chair Fastener

Transfer chair locking system

Multi Chair Fastener part no. 21-0180-006



Multi Chair Fastener part no. 21-0180-006

Multi Chair Fastener

One lock for multiple chairs. Maximum safety, maximum practicality.

With the revolutionary Multi Chair Fastener locking system – 10G certified – any chair, whether motorised or not, can be locked inside the ambulance thanks to the multi-compatibility of the locking. Practical, intuitive and safe.

Which chairs is the Multi Chair Fastener compatible with?

The new locking system can be used with all Ferno chairs, but not only. It is compatible with the following models:

**The entire Ferno Venice line: Venice Base, Venice Plus, Venice Power;
Ferno Ezglide Power;
Ferno 59-T;
The 42 line: the 42-E and 42-EV chairs;**

Do you have a chair from another brand? No problem;

The Multi Chair Fastener is also compatible with a wide range of chairs from different manufacturers.


WHY CHOOSE THE MULTI CHAIR FASTENER

It's quick and safe.

The Multi Chair Fastener allows locking the chair in a closed configuration inside the ambulance. In this way, it can be stored in a small space and in safe conditions: the 10G certificate guarantees its reliability. Finally, the restraint provides additional stability.

It is effective.

Storage is quick for all chairs. The vertical sliding locking bracket allows adjusting the height using the red handle. The locking bracket allows infinite levels of adaptability, depending on the height of the chair handle: compatibility with countless chairs of different brands is guaranteed.

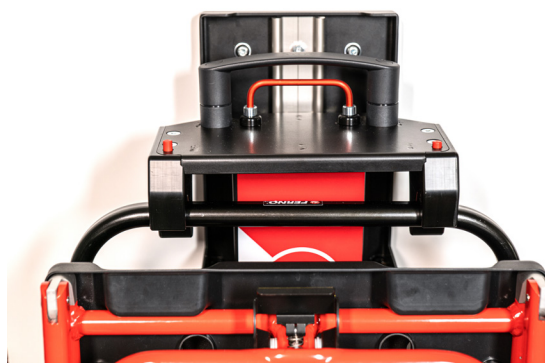

It's intuitive.

The notch on the locking bracket allows the chair handle to be easily anchored and the presence of two correct locking indicators ensures that the locking operation has been carried out correctly.

Zero effort.

The locking ensures that the chair can be stored in the ambulance by simply sliding the wheels into the special guides fixed to the floor. No need to lift the chair!

And to remove it from the support? Simply lift the red handle on the vertical locking bracket.



THE MAIN FEATURES OF THE MULTI CHAIR FASTENER

- Designed for all Ferno chairs and compatible with chairs of other brands;
- Vertical sliding locking bracket;
- Wheel guides fixed to the floor;
- Restraint included in the locking system;
- Ergonomic red handle for height adjustment;
- Indicators for correct locking;

2-year warranty

CERTIFICATIONS

- 10G certificate.
- UNI EN 1789 certificate.
- UNI EN ISO 9001:2018 certified
- CE Marking.

CHAIR LOCKING

The Multi Chair Fastener is compatible with a wide range of transfer chairs on the market.

1. Before storing the transfer chair in the Multi Chair Fastener locking system, close and lock it properly.
2. Push the chair's wheels into the relevant guides at the base of the Multi Chair Fastener system.
3. Position the chair so that it is under the handle of the Multi Chair Fastener system.
4. Place the chair correctly, paying the utmost attention to the track gears.
5. Grasp the main handle by pulling the red lever upwards (fig. 1), pushing the entire system downwards until you reach and lock the chair (fig. 2 /3).
6. Ensure correct locking by referring to the red indicators at the top of the locking system (fig. 3).

Note: when both red indicators are raised, the system is correctly locked.

7. Finally secure the chair with the restraint (fig. 4).

fig .1




fig .2



fig .3



fig .4

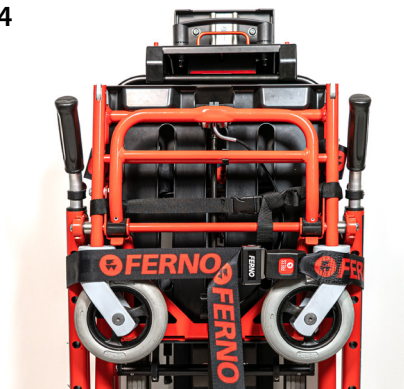



MULTI CHAIR FASTENER TECHNICAL SPECIFICATIONS

part no. 21-0180-006

Height	1154 mm
Width	631 mm
Depth	294.3 mm
Overall weight	11.5 kg

Multi Chair Fastener