

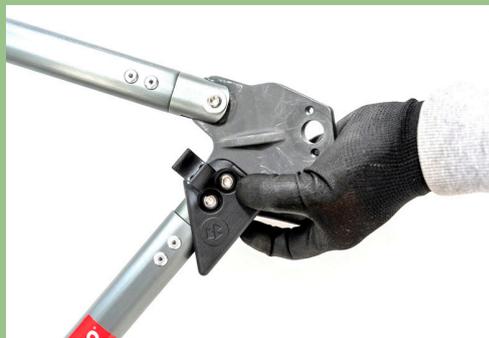
How to assemble your anti loss PIN

1

Initial Configuration for assembly.

Please note that the anti-loss PIN is to be installed on the V-Shaped Support marked with the **red-yellow color code**.

This is the one that features a through hole (non-threaded) on the joint plate connecting the two arms (unlike the blue-green support, which has a threaded hole instead).

2

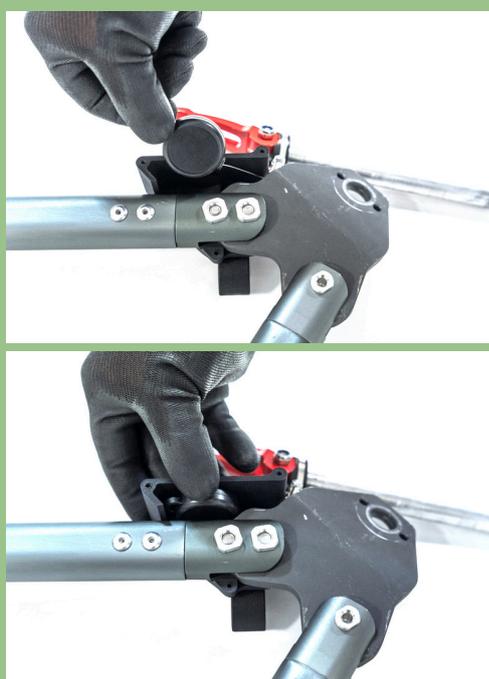
Take one half-shell of the anti-loss PIN and place it on the outer side of the V-Shaped Support.

Make sure to use the half-shell with the Ferno logo, and ensure the logo is facing outward.

The holes on the shell must align perfectly with the domed screw heads as shown in the picture.

3

Rotate the V-Shaped Support as shown in the reference image to facilitate the next steps.

4

Insert the retractable cable holder into its designated slot inside the shell. Make sure the wheel PIN remains outside the shell during placement.

5



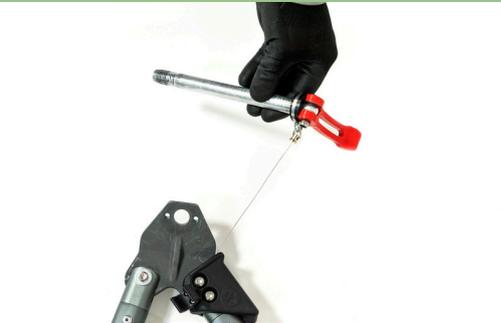
Position the second half-shell to enclose the cable holder completely and use the three included screws to fasten the two halves together. Tighten carefully to ensure a secure assembly without damaging components.



6



The anti-loss PIN is now correctly installed and ready for use.



7



When not in use, the PIN can be stored in the designated slot on the plastic casing. Make sure to place the PIN in the correct orientation.

ANTI LOSS PIN PROPER STORAGE



ANTI LOSS PIN WRONG STORAGE