DIRECTIONS FOR USE

EASY TRANSFER MATTRESS

UDI NO: 07350019593325

SAFE TRANSFER MATTRESS

UDI NO: 07350019593301

English: Directions for use. **EasyTransfer Mattress**
Deutsch: Bedienungsanleitung. **Easy Transfer Stretcher/Transfermatte**
Français: Mode d 'emploi. **Matelas de transfert Easy Transfer**
Español: Instrucciones de uso. **Colchoneta de camilla/traslado Easy Transfer**
Svenska: Bruksanvisning. **Easy Transfer Madrass**
Italiano: Istruzioni per l 'uso. **Materassino Barella/Trasporto Easy Transfer**
Português: Instruções de utilização. **Maca/Colchão de Transferência Easy Transfer**
Dansk: Brugsanvisning. **Easy Transfer bære/transportmadras**
Nederlands: Gebruiksaanwijzing. **Easy Transfer Brancard/Transfer Matras**
Suomi: Käyttöohje. **Easy Transfer Stretcher Mattress**
Norsk: Brukerveiledning. **Easy Transfer bære/transportmadrass**

Distributor:

Ferno Washington Inc.
70 Weil Way
Wilmington, OH 45177
USA

Phone: +1 937 382 1451

Fax: +1 937 382 6569

E-mail: info@ferno.com

www.ferno.com

Manufacturer:

AB Germa
Industrigatan 54-56
SE-291 36 Kristianstad
Sweden

Phone: +46 (0)44 12 30 30

Fax: +46 (0)44 10 31 79

E-mail: info@germa.se

www.germa.se

English:	Directions for use	Page	3
Deutsch:	Bedienungsanleitung	Seite	5
Français:	Mode d'emploi	Page	7
Español:	Instrucciones de uso	Página	9
Svenska:	Bruksanvisning	Sida	11
Italiano:	Istruzioni per l'uso	Pagina	13
Português	Instruções de utilização	Página	15
Dansk:	Brugsanvisning	Side	17
Nederlands:	Gebruiksaanwijzing	Bladzijde	19
Suomi:	Käyttöohje	Sivu	21
Norsk:	Brukerveiledning	Side	23

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1. Warning and caution statements

CAUTION

- Avoid handling the mattress next to sharp objects.
- Avoid handling the mattress next to open fire.
- On a current basis check that the mattress has no cracks.
- Own modifications on products not allowed – Your medical specialist to be responsible for any modification.
- When decided to take the product out of service and destroy – make sure to follow your local instructions or contact us for additional information.

Recommendation: Usable life of 3 years

2. Intended use

The **Easy Transfer Mattress** (Stretcher/Transfer Mattress) is intended to be used as a comfortable mattress on the stretcher during transportation. It is also a device intended to facilitate the transfer of the patient from one stretcher to another.(EN 1865 and EN 1789).

The mattress can be carried by two persons for a shorter distance, and a longer distance if two or more persons use additional carrying belts. You can pull the mattress on the ground by the handles or by connecting a carrying belt or rope.

The mattress consists of an outer shell of airtight, fully welded, material that is filled with a foam inlay and is fitted with handles on two or four sides for carrying or pulling.

3. Specifications

The Stretcher/Easy Transfer Mattress is tested in accordance with EN 1021-1
The Stretcher/Easy Transfer Mattress is in conformity with EU - MDR 2017/745 Class 1 concerning Medical Devices and EN 1865, EN 1789.

Temperature range: -30°C to +70°C

Parts/Materials

Outer shell: PVC coated fabric (flame retardant)

Filling material: Foam of polyester

Plate in polypropene

Dimensions	Length (cm)	Width (cm)	Height (cm)	Weight (kg)	N °of handles
Easy Transfer Mattress 10	185	50	6	4,7 / 5,0	10
Safe Transfer Mattress 12	190	50	6	4,8 / 5,0	12

4. Description

The Easy Transfer Mattress is equipped with 5 handles at each side.

The mattress consists of an outer cover of hard-wearing PVC coated polyester weaved canvass with a foam filling.

5. Operating instructions

1. Put the mattress on the stretcher
2. Place the restraint system shoulder belts from the stretcher through the hole in the mattress upper part under pillow part. Holes are found under the pillow. NOTE 4 point safety belt, see pictures
3. Place the injured patient in the medically desired position on the Ferno Transfer Mattress.
4. Close the belts of the stretcher.

5. Check that the unit is stable and comfortable for the patient.
6. The injured patient is ready to be transported.
7. The Ferno Stretcher/ Easy Transfer Mattress can also be used as a stretcher mattress. Its dimensions allow it to fit any commonly sold stretcher.
8. The Ferno Stretcher/Easy Transfer Mattress can easily be slid over to another stretcher or trolley at the hospital or at change of transportation.

6. Cleaning and Disinfecting

At a regular basis, the Stretcher/Transfer mattress should be wiped with a moist cloth and dried, if dirty, the mattress may be washed with soapy water and rinsed with clean water.

Allow the mattress to dry before storage.

If the transfer mattress needs disinfection first clean the mattress carefully and let it dry. Pour a solution containing 70 % alcohol or the disinfectant called LYSETOL AF on a clean cloth. Smear the transfer mattress with the cloth.

7. Repair of Easy Transfer Mattress

If damage occurs repair as follows:

- a) Find a patch in a suitable size and put glue on both the patch and the damaged spot on the transfer mattress.
- b) Wait for the glue to become dust dry (approx. 2 min.) and place the patch on the damaged spot.
- c) Press firmly the whole area of the patch, so that there is a good contact and all air is pressed out under the patch.

The repair will receive full strength after 24 hours.

8. Storage

The Easy Transfer Mattress is often positioned on top of the stretcher with a covering blanket.

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Report no. :

(To be assigned in by
Manufacturer)

Please print or type or fill in by MS Word, all sections a. – i. Fax or e-mail to your supplier. Please use repair order when reporting repairs not associated with complaints.

For claims please specify :

Purchase Date

Description of product

Description of Incident/Problem: 1) What, 2) How?

When? In connection with: 1) transport, 2) unpacking, 3) preparing, 4) test, 5) cleaning, 6) normal use, 7) training, 8) disassembly, 9) assembly, 10) storage, 11) other:

Info on the product, prior to problem,

Storage: 1) stock, 2) site of use, 3) packed, 4) unpacked, 5) softpack, 6) other:

Used cleaning and disinfecting agents/Sterilizing method:

Pattern of use: 1) stand-by for emergency, 2) routine use, 3) stand by, 4) other:

Environment: 1) Indoor, 2) Outdoor, 3) Heat, 4) Cold, 5) Humidity, 6) Sunlight, 7) Gases, 8) other:

Which **actions** have been: a) taken, b) planned by the reporter or user at the site of incident?

1) examination of the product on site, 2) replacement of the product, 3) other:

Suggested actions by the distributor/complainant

Returning of samples/faulty product(s),

- ¹⁾ 1) Already shipped/Will be shipped to _____,
- ²⁾ 2) Pending request from manufacturer,
- ³⁾ 3) Product/parts not available:

Date:

We acknowledge the receipt of this report by manufacturer and will start processing it immediately. Please use the assigned complaint report number for further reference.

Your contact person in this matter is: _____

Name:

Direct Tel.

Direct Fax.

Init/Date: