AB GERMA®

INSTRUCTIONS FOR USE STEP BY STEP

English: Instructions for use. **Step by Step Transfer**

Svenska: Bruksanvisning. **Step by Step Transfer**

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INSTRUCTIONS FOR USE

STEP BY STEP

English: These instructions for use may be updated without further notice. Copies of the current version are available from the manufacturer.

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1. Introduction

The Step by Step straps are designed for moving and evacuating patients who are lying on a transfer mattress or vacuum mattress.

In emergency situations where evacuation is necessary, the straps make patient handling easier and safer.

They help paramedics move patients more efficiently across flat surfaces, down stairs, and over obstacles, while improving ergonomics and safer transfer for both the patient and the medical staff.

The straps can also be used when transferring a patient onto a stretcher for ambulance transport.

2. Precautions

- Operators must receive proper training and have experience using the Step by Step before use.
- When moving a patient down stairs, always transport the patient's feet first.
- Regularly inspect the Step by Step straps, webbing and seams for signs of wear or damage.
- **The Step by Step must not be used in CT, MRI, or X-ray diagnostic equipment.**
- Modifications to the product are not permitted. Any approved modifications are the responsibility of the medical specialist.
- When the product is taken out of service and disposed of, follow local regulations and disposal procedures.

3. Description

Step by Step comes in two versions:

Step by Step Easy Transfer:

- Longer sliding strap for head end than foot end.
- Four safety belts
- In blue/orange

Step By Step Transfer Black II

- Sliding straps are the same length.
- Sliding straps with handles
- Three safety belts
- Black

- Step by Step Transfer is equipped with sliding straps (also functioning as control straps) for each side of the vacuum or transfer mattress (head and feet). The storage bag also acts as a feet protection
- Step by Step also have safety belts for securing the patient on a transfer mattress.
- Materials on the safety belts, sliding straps, and bag are polyester and the carabiners are metallic.
- When not in use the Step by Step Kit is stored in the storage bag that takes up a minimal amount of space and also has the function of a feet protection.

Maximum load: 250 kg

4. Operating the Step By Step

INSTRUCTIONS FOR USE STEP BY STEP

1. Open the bag and remove the sliding straps and the safety belts.
2. Attach one sliding strap (the longer for Step by Step Easy Transfer) to the last handles at the head end of the vacuum or transfer mattress.
 - The carabiner can either be attached directly around the handle or preferably to the rings mounted on the strap.
 - Adjust the strap length to a comfortable working position by pulling the strap through the mattress handle and attaching the snap hook to one of the rings on the strap.
3. If the transfer mattress does not have safety straps, attach the patient safety belts to the side handles of the mattress.
 - Fasten each belt by looping it around the handle using the sewn loop on the belt.

Note: ***The upper belt should be positioned as close to the armpits as possible.
The middle belt should be placed high on the thighs, below the hips.
The lower belt should secure the lower legs.***

4. Attach the strap connected to the bag to the last handle at the foot end of the mattress.
 - If the patient is short, the foot bag can be tightened using the ring on the fastening strap.
5. Attach the remaining sliding strap to the last set of handles at the foot end of the vacuum or transfer mattress.
 - Adjust the strap length in the same way as for the head strap.
6. Place the patient on the vacuum or transfer mattress.
 - Put the patient's feet into the bag and ensure that the foot strap is positioned underneath the feet.
 - Tighten the drawstring lock around the ankles to prevent the feet from slipping out of the bag.
7. Once the patient is securely positioned on the mattress, transport can begin.

Note: ***Always transport the mattress with the foot end leading.***

8. Both operators should hold the sliding straps firmly and adjust their grip and position for optimal ergonomics.
 - When moving down stairs, the operator at the foot end should carefully walk backwards down the stairs while maintaining communication with the operator at the head end.
 - The operator at the foot end controls the speed by slightly lifting the mattress. Lowering the foot end slightly will reduce the speed.
9. The operator at the head end should hold the head straps firmly to control the descent of the mattress.
 - At the same time, the patient's head should be slightly lifted to prevent it from hitting the stairs.

Note: ***Good communication between operators is essential throughout the entire procedure.
The patient should also be informed about what is happening during the evacuation.***

10. At stair landings, use the low-friction underside of the transfer mattress and pull the sliding straps to guide the mattress safely through turns.
11. Once at the bottom of the stairs, disconnect the Step by Step sliding straps before lifting the patient and mattress onto the stretcher.

5. Cleaning and Disinfection

Step by Step should be cleaned regularly by wiping it with a damp cloth and then drying it thoroughly.

If the product becomes dirty, it can be cleaned with soapy water and rinsed with clean water. When necessary, the safety belts and straps may be machine washed inside a laundry bag.

Note: *Remove the carabiners before machine washing.*

Always allow the Step by Step to dry completely before storage.

6. Maintenance and Repair

If damage occurs, repair or replace components as follows:

- Damaged carabiner: replace it with a new carabiner supplied by Germa.
- Damaged or worn patient safety belts: replace the belts with new ones from Germa.
-

Any safety belt or strap showing signs of wear or damage must be replaced for safety reasons.

7. Storage

The Step by Step should be stored in its bag under normal ambulance storage conditions.

8. Specifications

The Step By Step is an accessory to a medical device and is classified as Class I in accordance with Rule 1 of Annex VIII of MDR 2017/745.

Dimensions:

Length x Width x Height: see product information below
Weight: see product information below

Materials:

Part	Material
Straps & Belts	Polyester
Handles	PVC
Carabiner	Galvanized Steel

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INSTRUCTIONS FOR USE STEP BY STEP

NOTICE TO USERS AND PATIENTS

In the event of a serious incident related to the use of this device, it is imperative to take the following steps:

1. Report the Incident to the Manufacturer.

Contact the manufacturer of the device promptly. Ensure that all relevant details about the incident, including the circumstances and any observed effects, are provided to facilitate investigation and resolution.

2. Notify the Competent Authority.

The competent authority in the Member State where you are established must also be informed about the incident. This ensures compliance with regulations and enables authorities to take any necessary actions to safeguard public health.

Your prompt reporting helps improve device safety and ensures appropriate measures are taken to prevent future occurrences. Thank you for your cooperation.